



advantage
solutions

Alpha Pro Tech N-95 Particulate Respirator

PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®


Donning Instructions



Step 1. Open Mask

Grasp nose piece and chin piece and snap open to activate the Magic Arch®. A "pocket" is formed in the mask.

Note: Mask must be fully open.

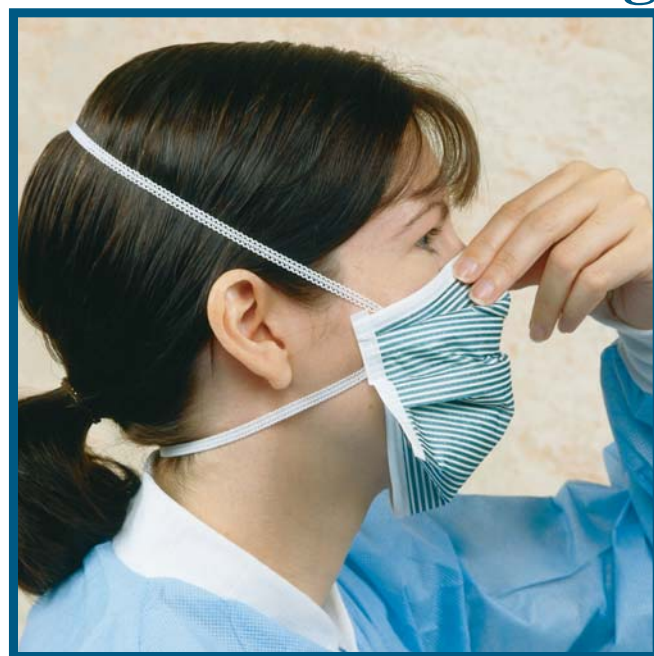


Step 2. Conform to Face

Place chin in pocket created, place bottom headband below ears and top band at crown of head. Conform nose piece to nose and facial contours.



Step 3. Fit to Face

Holding mask at nose piece, gently pull down under chin so mask fits snug against face.

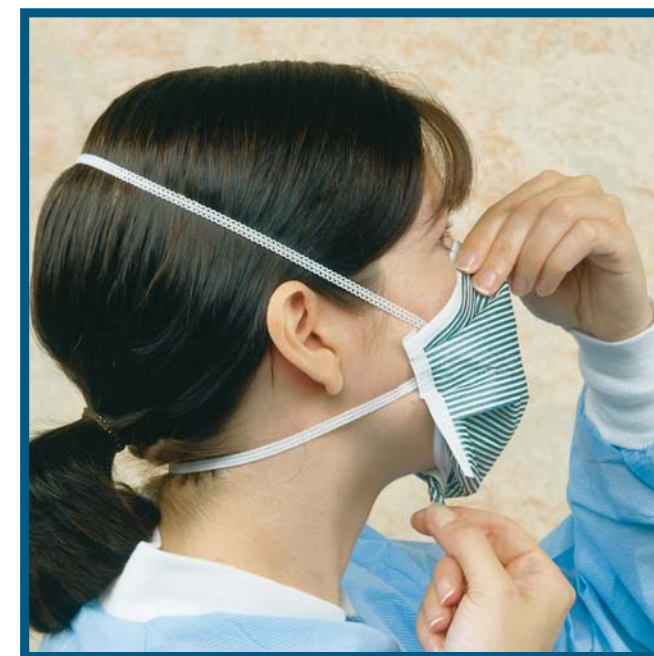

Step 4. Positive Facial Lock®

Pinch any excess material together under the chin and twist the Twist Seal® chin piece. Perform Fit Check (see below*).

***Fit Check** - Place both hands along the edges of the mask and exhale. If air escapes at your nose, adjust the nose piece. If air leaks at the edges, untwist the Twist Seal chin piece and repeat Step 3.

Warning

1. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards.

2. Follow all instructions and warnings on the use of this respirator. Failure to do so will reduce respirator effectiveness, wearer protection and may result in sickness or death. Wear during all times of exposure. For additional information refer to your supervisor and/or see dispenser box.

3. Before use, the wearer must be trained by the employer in proper respirator use in accordance with applicable safety and health standards for the contaminant and exposure level in the assigned work area.

Positive Face Lock, Magic Arch and Twist Seal are registered trademarks of AlphaProTech, Inc.

Reorder Part No. 695

AlphaProTech
INCORPORATED



RT Technologies Inc. | 2391 Briarleigh Way | Dunwoody, GA 30338 | 770-332-0092 | contact@rtlasersafety.com